

All imaging done at Western Radiology and Advanced Diagnostic Imaging can be viewed online via IntelViewer and IntelConnect.


? – What is the difference between IntelViewer and IntelConnect?

A – IntelViewer is a program that runs on your computer, IntelConnect is a web-page access to the same information, commonly referred to as a 'portal'.


To setup access to your patient's images online please follow the steps:

IntelViewer setup


1. In IntelViewer click 'EDIT BOOKMARKS'

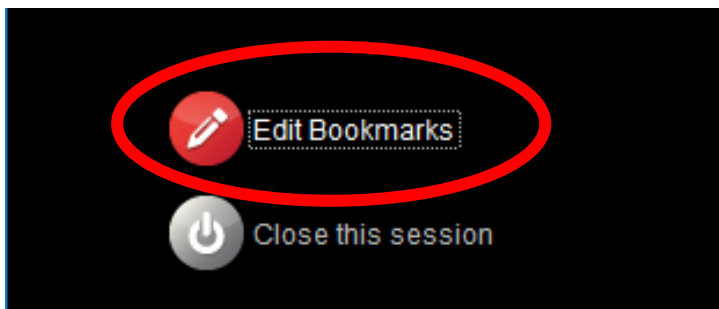

2. Click 'ADD'

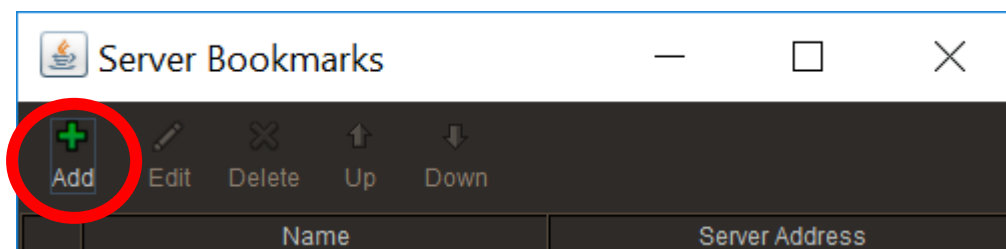

3. Insert server address as shown below, check both boxes below address field and type "Western Radiology" in the name field. Then click ok.

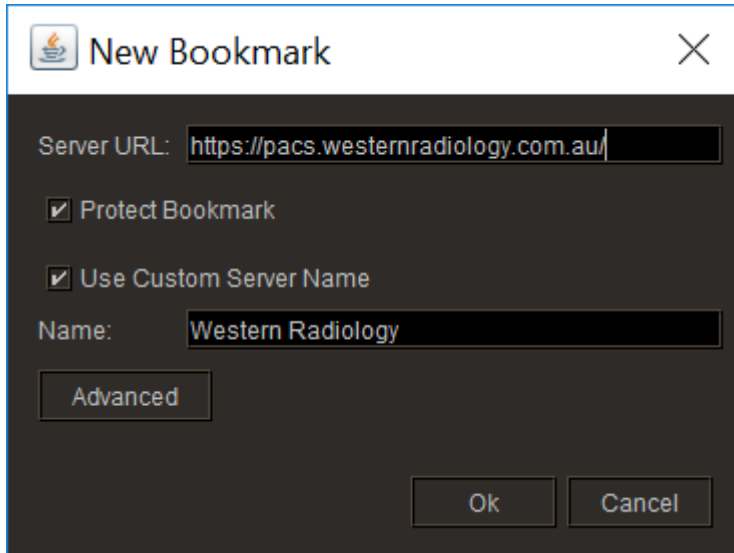

New Bookmark


Server URL:

Protect Bookmark


Use Custom Server Name

Name:

4. IntelViewer will return to the bookmarks screen. Press "Close" at the bottom.

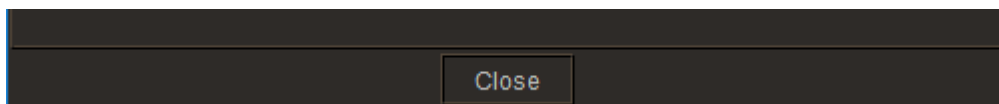

5. You will now be returned to the main IntelViewer screen and Western Radiology will be present as a bookmark. In the Username and Password fields input the information provided to you for this.

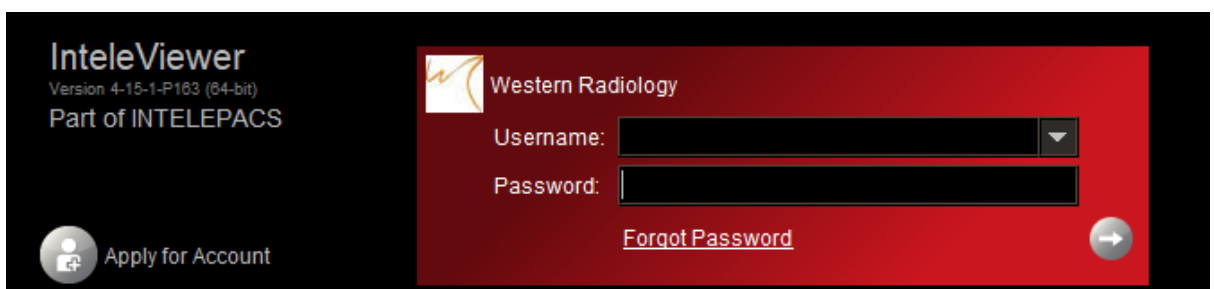

IntelViewer
Version 4-15-1-P183 (64-bit)
Part of INTELEPACS

Western Radiology

Username:

Password:

[Forgot Password](#)

6. Password autosave – To save your Password so that you don't need to enter it every time you want to use IntelViewer; at the main screen go to Utilities menu then click Automatic Login and enter your password one last time.



Searching for patients:


Select '**Break Glass**' to view **patients referred by others**. (Will need full name and DOB of patient).

IntelConnect setup

To setup access to our 'portal' via the web (IntelConnect) follow these steps:

1. Obtain **username & password** from reception marketing manager.
2. Go to <https://pacs.westernradiology.com.au/> and bookmark the page
3. Enter provided username & password and press login. Your web-browser should prompt you to **save password**, click **yes**.
4. The default view will normally present you with little or no results. Make sure you have the best view setup, with most recent studies on top. Here's how to set it up...


The default view...

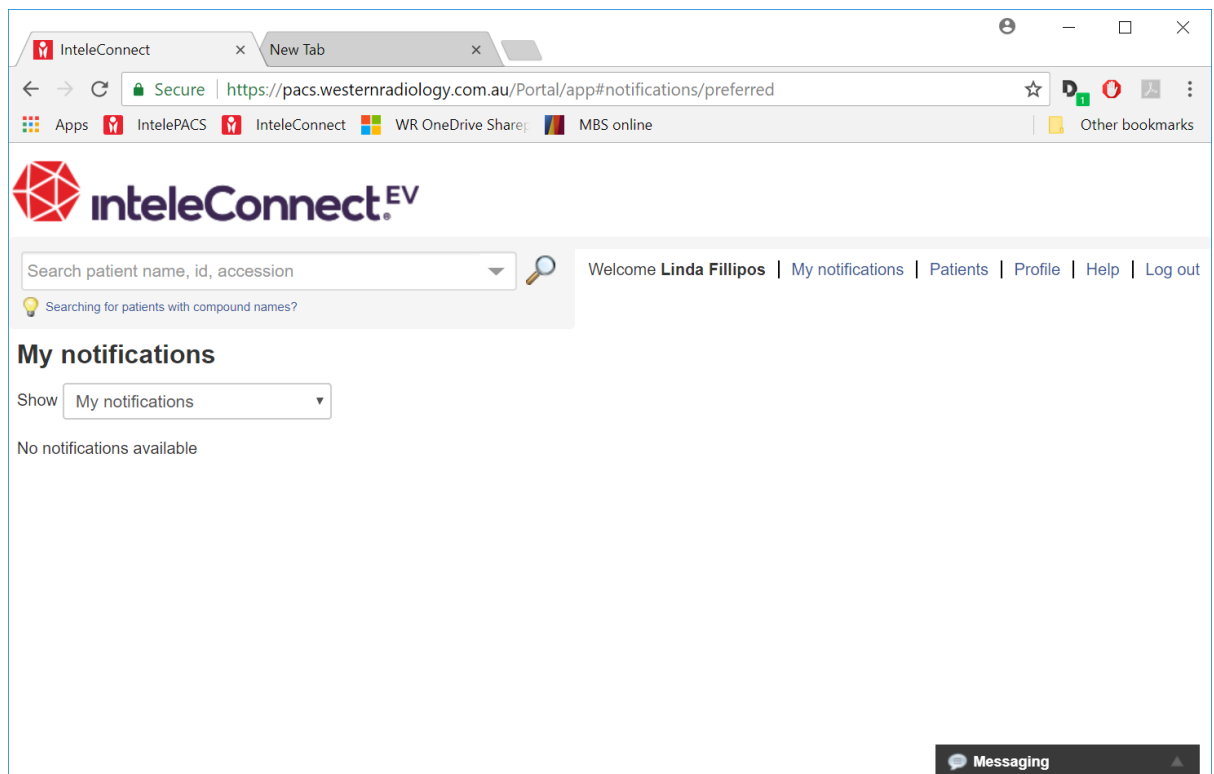

The screenshot shows a web browser window with the IntelConnect EV portal. The address bar displays the URL <https://pacs.westernradiology.com.au/Portal/app#notifications/preferred>. The page header includes the IntelConnect EV logo and a search bar for patient information. A navigation menu at the top right shows the user is logged in as Linda Fillipos, with links for My notifications, Patients, Profile, Help, and Log out. The main content area is titled "My notifications" and shows a dropdown menu set to "My notifications" with the message "No notifications available". A "Messaging" button is visible in the bottom right corner.


Changes to be made:

Step 1 Click "Patients"

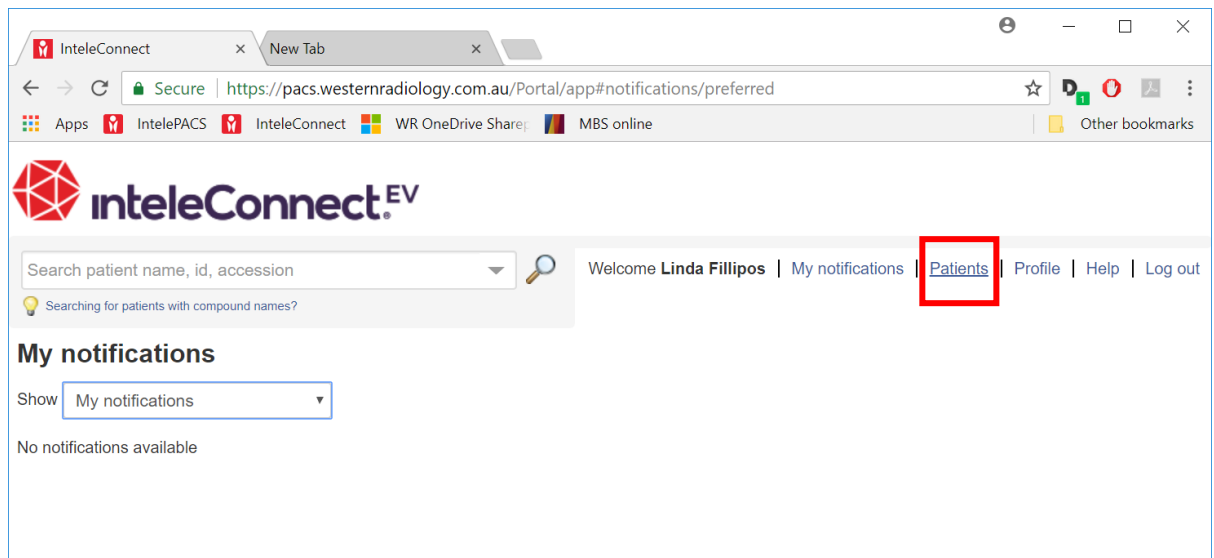

This screenshot is identical to the previous one, but the "Patients" link in the top navigation menu is highlighted with a red rectangular box, indicating the step to be taken.

Step 2 Click “My patients” then “Edit” next to “My patients” below

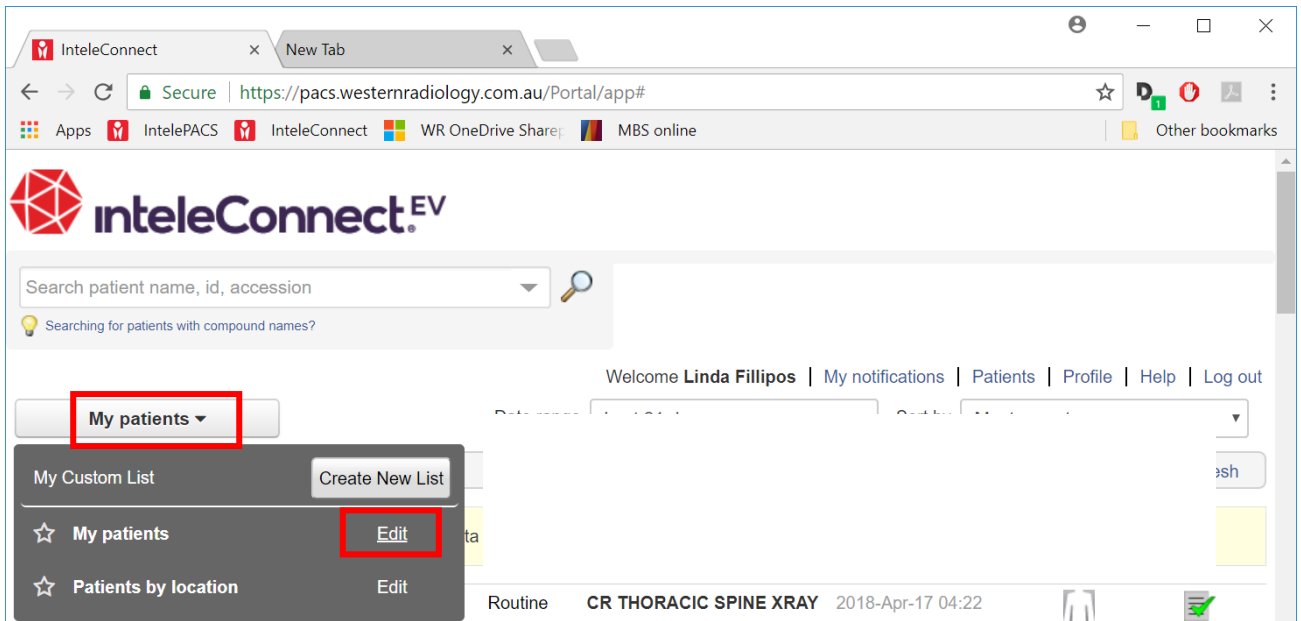

Step 3 A settings box will appear, change the name to “My patients last 31 days”. On “Date range” click and select “Last 31 days”. Under “Patients”, select “All” button.

Edit List Parameters

List name
My patients last 31 days

Show in worklist toolbar

Date range: Last 31 days

Exam Status:

Modality:

Patients: All Only those I referred


Discrepancies & critical results: Impression discrepancies Critical results

Sorting: Most recent

Organizations:

Locations:

You should now see this view. This should be what you see each time you log in from now on.

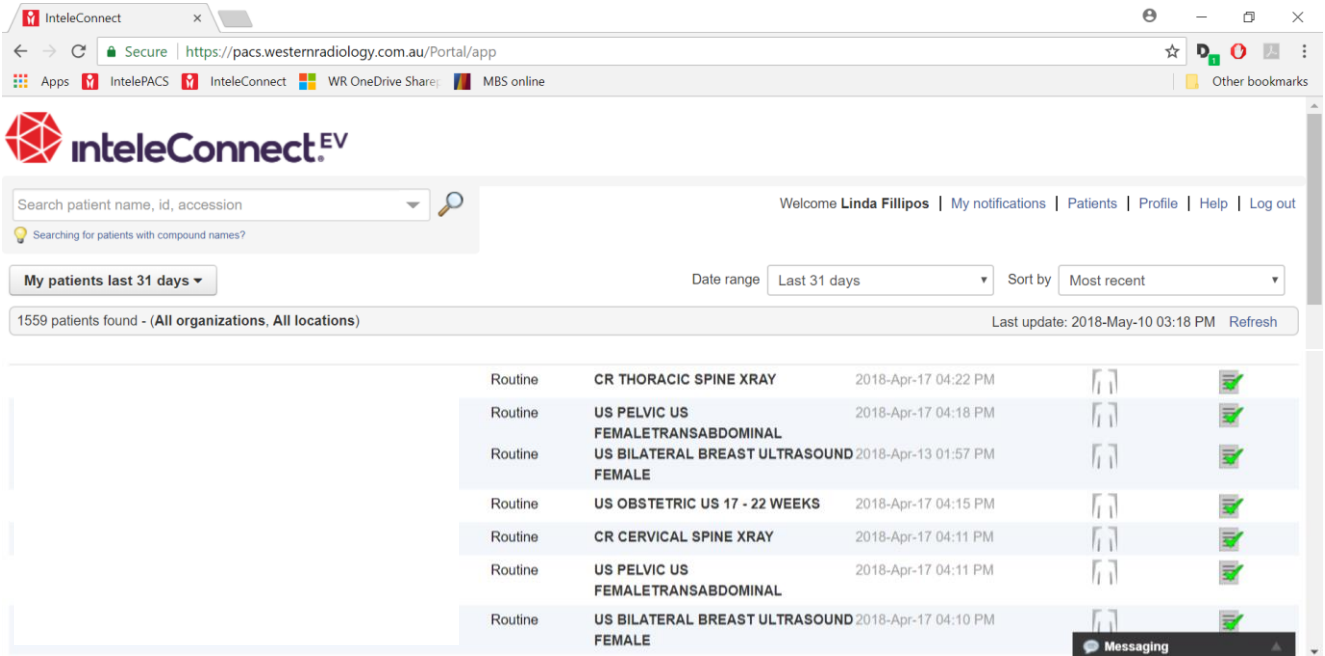

The screenshot shows the IntelConnect web application interface. At the top, there is a search bar for patient name, ID, or accession number. Below the search bar, there are filters for 'My patients last 31 days' and 'Date range' set to 'Last 31 days'. The interface displays a list of 1559 patients found, with the last update being 2018-May-10 03:18 PM. The patient list includes columns for 'Routine', 'Procedure Name', and 'Date/Time'. A 'Messaging' button is visible at the bottom right of the patient list.

Routine	Procedure Name	Date/Time	View	Check
Routine	CR THORACIC SPINE XRAY	2018-Apr-17 04:22 PM		
Routine	US PELVIC US FEMALETRANSABDOMINAL	2018-Apr-17 04:18 PM		
Routine	US BILATERAL BREAST ULTRASOUND FEMALE	2018-Apr-13 01:57 PM		
Routine	US OBSTETRIC US 17 - 22 WEEKS	2018-Apr-17 04:15 PM		
Routine	CR CERVICAL SPINE XRAY	2018-Apr-17 04:11 PM		
Routine	US PELVIC US FEMALETRANSABDOMINAL	2018-Apr-17 04:11 PM		
Routine	US BILATERAL BREAST ULTRASOUND FEMALE	2018-Apr-17 04:10 PM		